

INSTRUCTION FOR USE

EUGENOL

Eugenol is a dental material used in combination with Zinc Oxide to prepare temporary cement or sedative dental dressings. It is intended for use as a temporary filling material or cavity dressing and as a lining material under non-composite resin restorations.

COMPOSITION

Eugenol

INTENDED USE

Eugenol is intended for use in combination with Zinc Oxide as a temporary filling material, temporary dressing material, or cavity lining material in dental procedures.

INDICATIONS

- Use with Zinc Oxide for preparation of temporary filling materials
- Sedative temporary dental dressings
- Cavity lining under non-composite resin restorations
- Temporary restoration of teeth

CONTRAINDICATIONS

Eugenol is contraindicated for:

- Patients with known hypersensitivity or severe allergic reactions to Eugenol or any related ingredients
- Use beneath composite resin restorations due to inhibition of resin polymerization

INSTRUCTIONS FOR USE

1. Dispense the required quantity of Eugenol onto a clean mixing pad.
2. Add Zinc Oxide powder gradually and mix thoroughly using a suitable spatula until the desired consistency is obtained.
3. Prepare a thick paste suitable for the intended clinical use.
4. Apply the prepared material into the cavity as:

- A temporary filling material
- A sedative dressing for painful teeth
- A cavity lining under non-composite restorations

Adapt and contour the material as required.

Do not return dispensed material to the container.

Material once dispensed, mixed, or used must not be reused.

WARNINGS

- Avoid contact with skin, eyes, and oral soft tissues.
- Contact with skin may cause irritation, burns, or allergic hypersensitivity reactions.
- If skin contact occurs, immediately remove the material using cotton and alcohol, then wash thoroughly with soap and water.
- If skin rash, sensitization, or other allergic reactions occur, discontinue use and seek medical advice.
- If the product comes into contact with eyes, rinse immediately with plenty of clean water and seek medical attention.
- Do not ingest the product.

PRECAUTIONS

- Avoid direct contact with eyes and soft tissues.
- Eugenol may inhibit the polymerization of composite resin materials and therefore should not be used beneath composite restorations.

- Use only as directed.
- Use appropriate protective measures during handling.

PRESENTATION

Ref No.: PL-0108- Eugenol – 1 × 15 ml Glass Bottle

Ref No.: PL-0237- Eugenol – 1 × 100 ml Metal Bottle

STORAGE CONDITIONS

Store at temperatures between 10°C (50°F) and 24°C (75°F).

Keep the container tightly closed after use.

Keep away from moisture and direct sunlight.

Shelf life is not affected after opening if stored in a cool and dry place.

SHELF LIFE

3 years from the date of manufacture.

DISPOSAL

Dispose of product contents and packaging according to local regulations and applicable country-specific requirements.

CONTACT

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.


The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel


 CAUTION  CATALOGUE NUMBER

 KEEP AWAY FROM SUNLIGHT

 KEEP OUT OF THE REACH OF CHILDREN


 DO NOT USE IF PACKAGE IS DAMAGE


 KEEP DRY  MEDICAL DEVICE


 Symbol for "CONSULT INSTRUCTION FOR USE"

 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"

 Symbol for "NON STERILE"

 Symbol for "BATCH CODE"

 Symbol for "USE BY"

Document No	PL-IFU-030
Revision	02
Date	19-11-2025