

INSTRUCTION FOR USE

Preva Bond 7

(Light-curing single-component self-etch universal bonding agent)

Preva Bond 7 is a self-etch, single-bottle adhesive system designed to simplify bonding procedures. It combines etching, priming, and bonding functions in one solution. The formulation provides strong adhesion to enamel and dentin with compatibility for light-cured composite materials.

COMPOSITION

Dimethacrylate oligomers (Bis-GMA, HEMA, etc.), Colloidal silica filler, Functional monomer modifiers, Photoinitiator, Stabilizers, Organic solvents.

INDICATIONS FOR USE

- Direct composite restorations (Class I-V)
- Bonding of composite resin to enamel and dentin
- Repair of composite restorations
- Bonding of core build-up materials
- Cementation of indirect restorations (with compatible resin cements)
- Adhesive procedures in restorative dentistry

INSTRUCTIONS FOR USE

Tooth Preparation

- Clean tooth surface thoroughly.
- No separate etching is required (self-etch system).
- Lightly air-dry; avoid desiccation of dentin.

Application

- Dispense a small amount of Preva Bond 7 into a dispensing well.
- Apply generously using a microbrush to enamel and dentin.
- Actively scrub for 20 seconds.

Air Thinning

- Gently air-dry for 5-10 seconds to evaporate solvents.
- Ensure a thin, uniform adhesive layer.

Light Curing

- Light-cure for 10-20 seconds using a dental curing unit (≥ 1000 mW/cm²).

Restoration

- Proceed with composite placement or indirect restoration cementation.

WARNINGS & PRECAUTIONS

- For professional dental use only
- Flammable due to solvent content
- Avoid contact with skin, eyes, and oral soft tissues
- May cause allergic skin reactions in susceptible individuals
- Ensure proper isolation to avoid contamination
- Do not mix with other adhesive systems
- Use in a well-ventilated area
- Keep tightly closed after use to prevent solvent evaporation

CONTRAINDICATIONS

- Patients with known hypersensitivity to methacrylate resins
- Direct pulp exposure (without protective liner)
- Not recommended for use on untreated contaminated surfaces

ADVERSE EFFECTS

- Possible allergic reactions to methacrylate components
- Temporary irritation if improperly applied
- Post-operative sensitivity in rare cases

STORAGE CONDITIONS

- Store at 2-25°C
- Keep away from direct sunlight and heat sources
- Close bottle tightly immediately after use
- Do not freeze

REGULATORY NOTE

Preva Bond 7 is a light-curing universal adhesive system intended for professional restorative dental use only. Compatibility with resin-based restorative materials is required for optimal performance.

PRESENTATION

Ref.No: PL-0154- Preva Bond 7, Applicators, Dis. Mixing well.

SHELF LIFE

3 years from the date of manufacturing.

DISPOSAL

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS















Prevalent Healthcare LLP, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "CAUTION"
-  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"
-  Symbol for "TEMPERATURE LIMITATION"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
-  Symbol for "NON STERILE"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "MEDICAL DEVICE"
-  Symbol for "KEEP DRY"
-  Symbol for "USE BY"
-  Symbol for "BATCH CODE"
-  Symbol for "CATALOGUE NUMBER"
-  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "FLAMMABLE MATERIAL"

Document No	PL-IFU-072
Revision	02
Date	10-05-2025