

INSTRUCTION FOR USE

Preva Etch

(37% Phosphoric Acid Etchant Gel)

Preva Etch is a blue-colored, viscous gel formulated to provide excellent control and visibility during application. It ensures effective surface conditioning for improved adhesion of dental materials.

COMPOSITION

Phosphoric acid (37%), Glycerin, Solvent, Purified Water, Fillers and Pigments.

INDICATIONS FOR USE

- Etching of enamel for composite restorations
- Etching of dentin (when indicated)
- Bonding procedures for orthodontic brackets
- Surface preparation for sealants and adhesive systems

INSTRUCTIONS FOR USE

Preparation

- Clean and isolate the tooth surface.
- Ensure the area is free from contaminants (saliva, oil, debris).

Application

- Apply Preva Etch directly to the enamel surface using a dispensing tip.
- For enamel: etch for 15–30 seconds
- For dentin: etch for 10–15 seconds

Rinsing

- Rinse thoroughly with water for at least 15–20 seconds to remove all gel residues.

Drying

- Gently air-dry.
- Enamel should appear frosty white.
- Dentin should remain slightly moist (do not over-dry).

Procead

- Apply bonding agent immediately as per manufacturer's instructions.

WARNINGS & PRECAUTIONS

- Contains phosphoric acid – corrosive
- Avoid contact with skin, eyes, and soft tissues
- In case of contact, rinse immediately with plenty of water
- Use rubber dam or proper isolation techniques
- Do not ingest
- For professional dental use only.

CONTRAINDICATIONS:

- Do not use in patients with known hypersensitivity to phosphoric acid
- Avoid use on exposed pulp tissue

STORAGE CONDITIONS

- Store at temperature between 15°C (59°F) to 25°C (77°F).
- Close the cap carefully after use.
- Keep away from moisture.
- Shelf life not affected after opening if stored in a cool dry place.

SHELF LIFE:

3 years from the date of manufacturing.

PRESENTATION

Ref: PL-0158 - Preva Etch Gel 3 x 5ml Syringes

DISPOSABLE CONDITIONS

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS


Prevalent Healthcare LLP, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

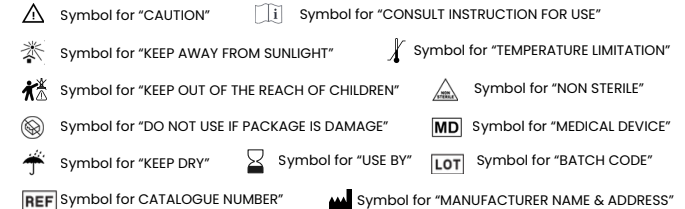

Document No	PL-IFU-018
Revision	02
Date	10-05-2024