

INSTRUCTION FOR USE

Preva Fill 2

(Glass Ionomer Restorative Cement)

Preva Fill 2 is a radiopaque Glass Ionomer Restorative Cement indicated for restoration of primary teeth, core build-up procedures, and restoration of Class III, Class V, and limited Class I cavities.

COMPOSITION

Powder: Fluoro Alumino Silicate Glass

Liquid: Polyacrylic Acid Liquid (Normal Viscosity)

INTENDED USE

Preva Fill 2 is intended for use as a permanent restorative material for both permanent and deciduous teeth.

INDICATIONS

- Restoration of primary teeth
- Core build-up procedures
- Restoration of Class III cavities
- Restoration of Class V cavities
- Restoration of limited Class I cavities

CONTRAINDICATIONS

Preva Fill 2 should not be used for:

- Direct pulp capping procedures
- Patients with known hypersensitivity or severe allergic reactions to any of the product components
- In rare cases, the product may cause sensitivity reactions in certain individuals. If such reactions occur, discontinue use and seek appropriate medical advice.

INSTRUCTIONS FOR USE

- Shake the powder bottle thoroughly before each use.
- Dispense one level scoop of powder and two drops of liquid onto the mixing pad.
- Using a plastic spatula, divide the powder into two equal portions.
- Mix the first portion of powder with all of the liquid for approximately 15 seconds.
- Add the remaining powder portion and continue mixing for an additional 20 seconds.
- Total mixing time: 35 seconds
- Transfer the mixed cement immediately to the prepared cavity using a suitable placement instrument.
- Shape and contour the restoration as required.
- Allow the material to set for approximately 1 minute 30 seconds to 2 minutes before finishing.
- Do not return dispensed material back into the container.
- Material once dispensed, mixed, or used must not be reused.

WARNINGS

- Avoid prolonged or repeated contact with skin and eyes.
- Avoid prolonged inhalation of powder.
- Do not ingest the product.
- Contact with skin may cause irritation, burns, or allergic hypersensitivity reactions.
- If skin contact occurs, immediately remove the material using cotton and alcohol, then wash thoroughly with soap and water.

- If skin rash, sensitization, or any allergic reaction develops, discontinue use and seek medical advice.
- In case of eye contact, rinse immediately with plenty of clean water and seek medical attention.
- Do not use in patients with known severe hypersensitivity to any product component.

PRECAUTIONS

- In case of accidental contact with oral soft tissues or skin, remove immediately using a cotton pellet or sponge soaked with alcohol and rinse thoroughly with water.
- If the material comes into contact with the eyes, flush immediately with plenty of water and seek medical advice.
- Use the product according to the recommended instructions only.

PRESENTATION

Ref No.: PL-0177- Preva Fill 2 - Powder – 20 g, Liquid – 15 ml, Spoon, Mixing Pad

SHADES

Available shades: A1, A2, A3

The restorative material is available in various tooth-colored shades. Use of the VITA Classical Shade Guide is recommended for appropriate shade selection.

STORAGE CONDITIONS

- Store at temperatures between 10°C (50°F) and 24°C (75°F).
- Keep container tightly closed after use.
- Protect from moisture.
- Shelf life is not affected after opening when stored in a cool and dry place.


SHELF LIFE

3 years from the date of manufacture.

DISPOSAL

Dispose of product contents and packaging according to local regulations and applicable country-specific requirements.

CONTACT

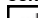












 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "CAUTION"  Symbol for "CATALOGUE NUMBER"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP DRY"  Symbol for "MEDICAL DEVICE"
-  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"
-  Symbol for "NON STERILE"
-  Symbol for "BATCH CODE"
-  Symbol for "USE BY"

Document No	PL-IFU-022
Revision	02
Date	10-08-2025