

## INSTRUCTION FOR USE

### Preva Pex

#### (Calcium Hydroxide with Iodoform Paste)

Preva Pex is a ready-to-use, radiopaque paste combining the antimicrobial properties of calcium hydroxide and iodoform. It is designed for easy placement into root canals and provides effective disinfection and temporary filling.

#### COMPOSITION

Iodoform, Calcium Hydroxide, Silicone Oil, Barium Sulfate

#### INDICATIONS FOR USE

- Temporary root canal filling in endodontic treatment
- Intracanal medicament between appointments
- Management of infected root canals
- Apexification procedures
- Resorption cases (internal and external)
- Pulpectomy procedures, especially in primary teeth

#### INSTRUCTIONS FOR USE

##### Preparation

- Isolate the tooth using a rubber dam.
- Perform cleaning and shaping of the root canal using standard endodontic procedures.
- Dry the canal using paper points.

##### Application

- Attach the provided dispensing tip to the syringe.
- Insert the tip into the canal, preferably short of the apex.
- Slowly inject the paste while withdrawing the tip to ensure complete canal filling.

##### Placement Verification

- Verify placement radiographically if required.

##### Temporary Sealing

- Seal the access cavity with a suitable temporary restorative material.

##### Removal (if required)

- Remove the paste using irrigation (e.g., saline or sodium hypochlorite) and mechanical instrumentation.

#### WARNINGS & PRECAUTIONS

- For professional dental use only
- Avoid extrusion beyond the root apex
- Avoid contact with eyes, skin, and oral soft tissues
- In case of accidental contact, rinse thoroughly with water
- Use appropriate protective equipment (gloves, mask, eye protection)
- Do not reuse disposable tips
- Ensure proper isolation to prevent contamination

#### CONTRAINDICATIONS:

- Do not use in patients with known hypersensitivity to any of the components (calcium hydroxide, iodoform, or excipients)
- Not intended for permanent root canal obturation
- Avoid use in cases where extrusion beyond the apex may cause complications

#### ADVERSE EFFECTS

- Possible irritation if extruded beyond apex
- Hypersensitivity reactions in susceptible individuals

#### STORAGE CONDITIONS

- Store in a cool, dry place
- Keep container tightly closed to prevent moisture contamination
- Protect from humidity and direct sunlight.

#### SHELF LIFE:

3 years from the date of manufacturing.


#### PRESENTATION

Ref.No: PL-0151 - Preva Pex - 3gm Syringe, Applicator Tips, Manual

#### DISPOSABLE CONDITIONS

Dispose of according to local regulations and facility guidelines.

#### CONTACT DETAILS

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.  
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

**This product is specially formulated for use in dentistry and to be used by qualified dental personnel**



Symbol for "CAUTION"



Symbol for "CONSULT INSTRUCTION FOR USE"



Symbol for "KEEP AWAY FROM SUNLIGHT"



Symbol for "TEMPERATURE LIMITATION"



Symbol for "KEEP OUT OF THE REACH OF CHILDREN"



Symbol for "NON STERILE"



Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"



Symbol for "MEDICAL DEVICE"



Symbol for "KEEP DRY"



Symbol for "USE BY"



Symbol for "BATCH CODE"



Symbol for "CATALOGUE NUMBER"



Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-019
Revision	02
Date	17-11-2024