

INSTRUCTION FOR USE

Preva ZOE+

(Polymer Reinforced ZOE Cement)

Preva ZOE+ is a polymer reinforced zinc oxide eugenol cement intended for temporary restorative applications. The unique polymer reinforcing agents provide excellent physical properties including high compressive and tensile strength, superior abrasion resistance, and low solubility. Its strength is comparable to Zinc Phosphate Cement.

COMPOSITION

Powder: Zinc Oxide, Fillers, Accelerators

Liquid: Eugenol, Fillers

INTENDED USE

Preva ZOE+ is intended for use as a temporary filling and intermediate restorative material.

INDICATIONS

- Intermediate restorative material (IRM) for Class I and Class II restorations
- Temporary restorations during emergency treatments
- Sealing the coronal portion of teeth undergoing endodontic treatment
- Use as a base under amalgam and other non-resin restorative materials

CONTRAINDICATIONS

- Preva ZOE+ is contraindicated in patients with known hypersensitivity or severe allergic reactions to eugenol or any of the product components.
- Preva ZOE+ should not be used as a liner or base under resin-based restorative materials due to the presence of eugenol, which may interfere with resin polymerization.

INSTRUCTIONS FOR USE

Preparation

- Clean and isolate the cavity preparation.
- Shake the powder bottle well before dispensing to ensure homogeneous density.

Mixing

- Dispense the required quantity of powder and liquid onto a clean mixing pad.
- Recommended powder-to-liquid ratio: 2 scoops of powder and 2-3 drops of liquid
- Mix thoroughly using a cement spatula for approximately 45-60 seconds until a uniform putty-like consistency is achieved.

Placement

- Carry the mixed material into the cavity preparation using a suitable instrument.
- Condense and contour as required.
- Allow the material to harden for approximately 4-6 minutes.

Disposal

- Do not return dispensed material to the original container.
- Material remaining after use must be discarded and not reused.

WARNINGS

- Avoid prolonged or repeated contact with skin, eyes, and oral soft tissues.
- Avoid prolonged inhalation of powder.
- Do not ingest the product.
- Contact with skin may cause irritation, burns, or allergic hypersensitivity reactions. In Case of Contact

Skin Contact:

- Immediately remove the material using cotton soaked in alcohol, then wash thoroughly with soap and water. If irritation, rash, or sensitization develops, discontinue use and seek medical attention.

Eye Contact:

- Rinse immediately with copious amounts of fresh water and seek medical advice.

PRECAUTIONS

- For professional dental use only.
- Do not use under resin-based restorative materials.
- Avoid contact with eyes and soft tissues.
- Do not invert the powder bottle while opening to prevent accidental excess dispensing.
- Avoid excessive force when dispensing liquid from the dropper bottle.
- Use appropriate personal protective equipment during handling.

PRESENTATION

Ref: PL-0355- Preva ZOE+ : 15 g Powder, 10 ml Liquid, Measuring Spoon, Mixing Pad

STORAGE CONDITIONS

Store at temperatures between 10°C (50°F) and 24°C (75°F). Keep containers tightly closed after use.

Protect from moisture and direct sunlight.

Shelf life is not affected after opening when stored in a cool and dry place.


SHELF LIFE

3 years from the date of manufacturing.

DISPOSAL

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com


Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.


This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel


 Symbol for "CAUTION"  Symbol for "CATALOGUE NUMBER"

 Symbol for "KEEP AWAY FROM SUNLIGHT"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP DRY"

 Symbol for "CONSULT INSTRUCTION FOR USE"

 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"

 Symbol for "MEDICAL DEVICE"  Symbol for "NON STERILE"

 Symbol for "BATCH CODE"

 Symbol for "USE BY"

Document No	PL-IFU-059
Revision	02
Date	10-05-2025