

INSTRUCTION FOR USE

PrevaHex

(2% Chlorhexidine Solution)

PrevaHex is a mint-flavoured aqueous solution containing chlorhexidine gluconate as the active antimicrobial agent. It provides broad-spectrum antibacterial activity and is used for oral cavity disinfection and adjunct plaque control.

COMPOSITION

Chlorhexidine Gluconate, Glycerin, Purified Water, Mint Flavoring Agent

INDICATIONS FOR USE

- Root canal irrigation and dentinal tubule disinfection
- Chemical cleansing of infected pulp tissue
- Adjunctive antimicrobial rinse during endodontic therapy
- Adjunct treatment for gingivitis and periodontal care
- Oral cavity disinfection prior to dental procedures

INSTRUCTIONS FOR USE

Preparation

- Ensure proper isolation of the treatment area when required.
- Shake bottle gently before use if recommended by manufacturer.

Application

- Apply PrevaHex using a sterile cotton pellet, microbrush, gauze, or applicator tip.
- Ensure even coverage of the target area (gingival margins, mucosa, or treatment site).
- Allow 30–60 seconds contact time for effective antimicrobial action.

Post-Application

- Excess solution may be gently air-dried or blotted with sterile gauze if required.
- Proceed with the intended dental procedure (restoration, scaling, impression, etc.).

Reapplication (if needed)

- May be reapplied during the procedure if contamination is suspected.

WARNINGS & PRECAUTIONS

- For professional dental use only
- Avoid contact with eyes and middle ear
- Do not inject or use internally beyond intended surface application
- May cause temporary staining of oral surfaces with repeated use
- Use in a well-ventilated clinical area
- Discontinue use if irritation or hypersensitivity occurs
- Keep out of reach of children

CONTRAINDICATIONS

- Do not use in patients with known hypersensitivity to chlorhexidine
- Not recommended for children unless directed by a dental professional
- Avoid use in patients with history of allergic reactions to oral antiseptics

ADVERSE EFFECTS

- Temporary mucosal irritation
- Rare allergic reactions
- Possible mild staining with repeated exposure

STORAGE CONDITIONS

- Store at room temperature (15–25°C)
- Keep container tightly closed
- Protect from heat and direct sunlight

PRESENTATION

Ref.: PL-0149- PrevaHex- 100ml Bottle

SHELF LIFE

3 years from the date of manufacturing.

DISPOSAL

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS














Prevalent Healthcare LLP, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "CAUTION"
-  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"
-  Symbol for "TEMPERATURE LIMITATION"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
-  Symbol for "NON STERILE"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "MEDICAL DEVICE"
-  Symbol for "KEEP DRY"
-  Symbol for "USE BY"
-  Symbol for "BATCH CODE"
-  Symbol for CATALOGUE NUMBER
-  Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-061
Revision	01
Date	10-05-2024