

INSTRUCTION FOR USE

PrevaLuxe

Prophylaxis Paste with 1.23% Fluoride Ion

PrevaLuxe Prophylaxis Paste with 1.23% Fluoride Ion is a spatter-free, medium grit cleaning and polishing paste, offered in multiple flavors. Specially formulated abrasives deliver efficient cleaning and a high-lustre finish. The non-spatter formula ensures quick clean-up.

COMPOSITION

Zirconium Silicate, Glycerin, Fumed Silica, Sodium Fluoride, Pigments and flavoring agent.

INTENDED USE

For professional dental prophylaxis involving cleaning and polishing of natural teeth and restorative surfaces.

INDICATIONS

- Removal of soft and hard surface plaque
- Elimination of extrinsic stains (coffee, tea, tobacco)
- Cleaning and polishing of restorations
- Removal of temporary luting material prior to final cementation
- Polishing tooth surfaces before whitening or after orthodontic treatment

INSTRUCTIONS FOR USE

1. Dispense PrevaLuxe Prophylaxis Paste into dappen dish or other suitable container. A saliva ejector may be helpful during the actual prophylaxis.
2. Fill rubber prophylaxis cup with paste. Before starting hand piece, carry paste to field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste be provided for each surface in order to obtain maximum cleaning effectiveness.
3. Run rubber cup at SLOW SPEED over each tooth surface for approximately 10 seconds. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.
4. A rubber cup or soft bristle brush may be used on occlusal surfaces.
5. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

WARNINGS & PRECAUTIONS

- Avoid eye contact. If exposed, rinse with plenty of water for 15 minutes. Seek medical attention if irritation persists.
- Ensure teeth are properly scaled and examined before use.
- Operator must use gloves, mask, and protective eyewear.
- For professional dental use only.

CONTRAINDICATIONS

Do not use in patients with known allergies or sensitivities to any of the ingredients.

PRESENTATION

1 X Jar of 200 gm

Ref:

- PL-0320 (Peppermint)
- PL-0321 (Strawberry)
- PL-0322 (Bubble Gum)
- PL-0323 (Caramel)
- PL-0324 (Mango)

SHELF LIFE

3 years from the date of manufacturing.

STORAGE CONDITIONS

Store at 10°C (50°F) – 25°C (77°F).

Keep container tightly closed. Protect from moisture.

Shelf life remains unaffected after opening if stored properly.

DISPOSAL

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS








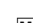

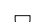



Prevalent Healthcare LLP, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "TEMPERATURE LIMITATION"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "NON STERILE"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "MEDICAL DEVICE"
-  Symbol for "KEEP DRY"  Symbol for "USE BY"  Symbol for "BATCH CODE"
-  Symbol for CATALOGUE NUMBER"  Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-098
Revision	01
Date	10-10-2025