



Back to Home



INSTRUCTION FOR USE

PrevaRelief

(Dentine Desensitizer)

PrevaRelief is a low-viscosity liquid formulation designed for professional dental use. It combines protein coagulation and resin infiltration mechanisms to provide immediate and effective relief from hypersensitivity. The formulation allows deep penetration into dentinal tubules, resulting in durable desensitization.

COMPOSITION

HEMA, Glutaraldehyde, Sodium Fluoride, Potassium Nitrate, Primer, Purified Water

INDICATIONS FOR USE

- Treatment of dentinal hypersensitivity due to exposed dentin
- Hypersensitivity associated with cervical abrasion, erosion, or gingival recession
- Post-scaling and root planing sensitivity
- Prior to or after restorative procedures
- Following tooth preparation

INSTRUCTIONS FOR USE

Preparation

- Clean the tooth surface and isolate the area.
- Gently air-dry; do not desiccate dentin.

Application

- Apply a small amount of PrevaRelief using a microbrush or applicator.
- Gently rub onto the exposed dentin surface for 30–60 seconds.

Drying

- Gently air-dry to evaporate the solvent.
- Do not rinse.

Repeat (if required)

- A second application may be performed for enhanced effect.

Proceed

- Continue with restorative procedures if indicated.

WARNINGS & PRECAUTIONS

- For professional dental use only
- Avoid contact with skin, eyes, and oral soft tissues
- Use rubber dam or proper isolation when possible
- In case of contact, rinse immediately with plenty of water
- Do not ingest or inhale vapors
- Use in a well-ventilated area
- Keep out of reach of children

CONTRAINDICATIONS

- Do not use in patients with known hypersensitivity to any of the components
- Do not apply on exposed pulp or near pulp tissue
- Avoid use on soft tissues

ADVERSE EFFECTS

- Temporary irritation if applied improperly
- Allergic or hypersensitivity reactions in susceptible individuals
- Pulpal irritation if used on deep cavities without protection

STORAGE CONDITIONS

- Store at room temperature (15–25°C)
- Keep container tightly closed
- Protect from heat and direct sunlight

PRESENTATION

Ref.: PL-0315- PrevaRelief- 5ml Bottle


SHELF LIFE

3 years from the date of manufacturing.

DISPOSAL

Dispose of according to local regulations and facility guidelines.

CONTACT DETAILS

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel



Symbol for "CAUTION"



Symbol for "CONSULT INSTRUCTION FOR USE"



Symbol for "KEEP AWAY FROM SUNLIGHT"



Symbol for "TEMPERATURE LIMITATION"



Symbol for "KEEP OUT OF THE REACH OF CHILDREN"



Symbol for "NON STERILE"



Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"



Symbol for "MEDICAL DEVICE"



Symbol for "KEEP DRY"



Symbol for "USE BY"



Symbol for "BATCH CODE"



Symbol for CATALOGUE NUMBER"



Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-066
Revision	01
Date	10-05-2024