



Back to Home

INSTRUCTION FOR USE



PrevaTemp

Self-curing, ready-to-use eugenol-free temporary filling material

PrevaTemp is a self-curing, ready-to-use eugenol-free temporary filling material based on zinc oxide and synthetic resins. It provides adequate strength yet is easily removable. The material adheres well to dentine with excellent marginal adaptation. Being hydrophilic, it rapidly cures with moisture in the oral cavity and simultaneously expands and hardens to provide a tight, effective marginal seal.

COMPOSITION

Calcium Sulphate, Zinc Sulphate, Zinc oxide, Natural Resin.

INDICATIONS

- Temporary filling of cavities
- Temporary sealing of medicaments
- Repair of lost fillings
- Restoration of deciduous (milk) teeth –all cavity classes

INSTRUCTIONS FOR USE

1. Using a suitable instrument, introduce PrevaTemp paste directly into the wet cavity.
2. Press and spread the material to fully fill the cavity and adapt it to the cavity walls.
3. Remove excess paste by wiping with a moist cotton pellet toward the cavity margins.
4. After setting, the hardened material may be removed using a rotary instrument or an appropriate hand instrument.
5. Do not fill very deep cavities up to the base; leave adequate space to avoid pressure on the pulp.

WARNING & PRECAUTIONS

- Avoid prolonged or repeated contact with skin or eyes. May cause irritation or allergic contact dermatitis.
- Avoid inhalation. Do not ingest.
- In case of skin contact, wipe thoroughly with cotton and alcohol, then wash with soap and water.
- If any allergic reaction, rash, or sensitization occurs, discontinue use immediately and seek medical attention.
- If the product enters the eyes, rinse immediately with plenty of water and consult a physician.
- Do not use in patients with known hypersensitivity to any of the ingredients.
- For professional dental use only.

CONTRAINDICATIONS

Do not apply directly on exposed pulp.

PRESENTATION

Ref: PL-0235 – 1 X 40 gm White Paste

Ref: PL-0236 – 1 X 40 gm Pink Paste

SHELF LIFE

3 years from the date of manufacturing.

STORAGE

- Store at 10°C – 25°C.
- Close the cap tightly after use.
- Protect from moisture.
- Shelf life is unaffected after opening if stored in a cool, dry place.

DISPOSAL

Dispose of according to local regulations applicable in the country of use.

CONTACT DETAILS

Prevalent Healthcare LLP, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.

Ph.: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

- Symbol for "CAUTION"
- Symbol for "CONSULT INSTRUCTION FOR USE"
- Symbol for "KEEP AWAY FROM SUNLIGHT"
- Symbol for "TEMPERATURE LIMITATION"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
- Symbol for "NON STERILE"
- Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "MEDICAL DEVICE"
- Symbol for "KEEP DRY"
- Symbol for "USE BY"
- Symbol for "BATCH CODE"
- Symbol for CATALOGUE NUMBER"
- Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-057
Revision	02
Date	19-11-2025