



Back to Home



## INSTRUCTION FOR USE

### PrevaSol

#### (Sodium Hypochlorite Solution)

Prevalent **PrevaSol** is a high-purity Sodium Hypochlorite Solution formulated for root canal irrigation. Infused with a subtle lemon aroma to reduce chlorine odor, it enhances patient comfort without compromising clinical efficacy. Available in 3% and 5% concentrations, it supports effective debridement and disinfection during endodontic procedures.

#### COMPOSITION

Sodium Hypochlorite, Purified Water, Flavoring Agent, Stabilizing Agents

#### INDICATIONS FOR USE

- Root canal irrigation during endodontic treatment
- Removal of organic tissue and debris
- Disinfection of infected root canals
- Canal debridement during instrumentation

#### INSTRUCTIONS FOR USE

1. Isolate tooth using rubber dam.
2. Prepare and access the root canal using standard endodontic procedures.
3. Irrigate canal using a side-vented irrigation needle.
4. Deliver solution slowly and passively without pressure.
5. Avoid binding the needle in the canal.
6. Use intermittently during instrumentation and as final rinse if required.

#### WARNINGS & PRECAUTIONS

- For professional dental use only
- Highly corrosive material
- Avoid extrusion beyond apex
- Avoid contact with skin, eyes, and soft tissues
- Do not mix with chlorhexidine or acidic solutions
- In case of contact, rinse immediately with water
- Use rubber dam isolation and protective equipment

#### CONTRAINDICATIONS

- Hypersensitivity to sodium hypochlorite
- Not to be used beyond root canal space
- Avoid use in open apex cases with high extrusion risk

#### ADVERSE EFFECTS

- Severe tissue irritation if extruded
- Chemical burns on soft tissue contact
- Pain and swelling in case of accidental extrusion

#### STORAGE CONDITIONS

- Store at room temperature (15–25°C)
- Keep container tightly closed
- Protect from heat and direct sunlight

#### PRESENTATION

Ref.No: PL-0127- PrevaSol 3%- 500ml Bottle .

Ref.No: PL-0128- PrevaSol 5%- 500ml Bottle.

#### SHELF LIFE

3 years from the date of manufacturing.

#### DISPOSAL

Dispose of according to local regulations and facility guidelines.

#### CONTACT DETAILS

**Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.  
Ph: +91 4931257954 Email: info@prevalentonline.com www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life.

The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

#### This product is specially formulated for use in dentistry and to be used by qualified dental personnel

- Symbol for "CAUTION"
- Symbol for "CONSULT INSTRUCTION FOR USE"
- Symbol for "KEEP AWAY FROM SUNLIGHT"
- Symbol for "TEMPERATURE LIMITATION"
- Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
- Symbol for "NON STERILE"
- Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
- Symbol for "MEDICAL DEVICE"
- Symbol for "KEEP DRY"
- Symbol for "USE BY"
- Symbol for "BATCH CODE"
- Symbol for "CATALOGUE NUMBER"
- Symbol for "MANUFACTURER NAME & ADDRESS"

Document No	PL-IFU-062
Revision	01
Date	10-05-2024