

## QuickZOE

### (Fast Set ZOE Cement)

QuickZOE is a fast-setting, easy-mix Zinc Oxide Eugenol (ZOE) cement designed for thermal insulation and temporary restorative applications. The material provides a soothing effect on the pulp and is intended for temporary dental procedures requiring reliable sealing and ease of manipulation.

#### COMPOSITION

**Powder:** Zinc Oxide

**Liquid:** Eugenol, Accelerators

#### INTENDED USE

QuickZOE is intended for use as a temporary filling and temporary restorative material in dental procedures.

#### INDICATIONS

- Temporary restorations
- Temporary cementation of crowns
- Cavity liner under most restorative materials

#### CONTRAINDICATIONS

- Patients with known hypersensitivity or severe allergic reactions to eugenol or any of the product components
- Use as a liner beneath resin-based restorative materials, as eugenol may interfere with polymerization

#### INSTRUCTION FOR USE

##### Mixing Procedure

- Dispense one scoop of powder and 3–4 drops of liquid onto a clean mixing pad.
- Mix thoroughly for approximately 45–60 seconds until a homogeneous putty-like consistency is obtained.

##### Application

- Place the mixed material into the prepared cavity or restoration area using a suitable instrument.
- Adapt the material properly and remove excess immediately.
- Allow the material to harden for approximately 11–12 minutes.
- Finish and trim the restoration using a tungsten carbide bur if required.

##### Important Handling Instructions

- Do not return dispensed material to the original container.
- Material once dispensed, mixed, or used must not be reused.
- Use clean and dry instruments during dispensing and mixing.

#### WARNINGS

- For dental professional use only.
- Avoid prolonged or repeated contact with skin, eyes, and oral soft tissues.
- Avoid prolonged inhalation of powder or vapors.
- Do not ingest.
- Do not use in patients with known allergies to any component of the material.

#### Skin Contact:

Contact may cause irritation, burns, allergic contact dermatitis, or hypersensitivity reactions. If contact occurs:

- Immediately wipe the affected area thoroughly with cotton and alcohol.
- Wash with soap and water.
- Discontinue use and seek medical attention if rash, irritation, or sensitization develops.

#### Eye Contact :

- If the product contacts the eyes: Rinse immediately with plenty of fresh water.
- Seek medical attention promptly.

#### PRECAUTIONS

- QuickZOE is a eugenol-based material and should not be used under resin-based restorative materials.
- Avoid contamination with moisture during storage and handling.
- Avoid contact with eyes and skin.
- Do not invert the powder bottle upside down while opening to prevent accidental spillage.
- Avoid excessive force when dispensing liquid from the dropper bottle.
- Close all containers tightly immediately after use.

#### PACKAGING

Ref. No.: PL-0354- QuickZOE- 15 g Powder, 10 ml Liquid, Measuring Spoon, Mixing Pad

#### STORAGE CONDITIONS

- Store at temperatures between 10°C and 24°C (50°F–75°F).
- Keep containers tightly closed after use.
- Protect from moisture, direct sunlight, and excessive heat.
- Store in a cool and dry place.


#### SHELF LIFE

3 years from the date of manufacturing.

#### DISPOSAL

Dispose of according to local regulations and facility guidelines.

#### CONTACT DETAILS


 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.  
Ph.: +91 4931257954 Email: info@prevalentonline.com  
www.prevalentonline.com


Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer. This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

### This product is specially formulated for use in dentistry and to be used by qualified dental personnel


 Symbol for "CAUTION"  Symbol for "CATALOGUE NUMBER"

 Symbol for "KEEP AWAY FROM SUNLIGHT"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP DRY"

 Symbol for "CONSULT INSTRUCTION FOR USE"

 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"

 Symbol for "MEDICAL DEVICE"  Symbol for "NON STERILE"

 Symbol for "BATCH CODE"

 Symbol for "USE BY"

Document No	PL-IFU-058
Revision	02
Date	10-05-2025