

Formacresol

(Dental Medicament)

Prevalent Formacresol is a dental medicament used for root canal cleansing, disinfection, and pulpal devitalization procedures. It is primarily indicated for use in vital pulpotomy procedures and in the management of infected root canals due to its antibacterial and fixative properties.

COMPOSITION

Formaldehyde, Cresol, Glycerin, Isopropyl Alcohol

INDICATIONS

- Root canal cleansing and disinfection
- Root canal dressing
- Vital pulpotomy procedures in primary teeth
- Management of infected root canal conditions

CONTRAINDICATIONS

- Do not use in patients with known hypersensitivity or allergy to formaldehyde, cresol, or any other components of the formulation
- Do not use in patients with a history of severe allergic reactions to any ingredients of the product
- Do not use for applications other than intended use

WARNINGS

- Contains formaldehyde which may be toxic if inhaled or ingested
- May cause irritation, chemical burns, or hypersensitivity reactions upon contact with skin or mucosal tissues
- Avoid prolonged or repeated exposure
- Avoid inhalation of vapors
- Do not swallow the product
- Keep out of reach of children

In case of accidental exposure:

Skin contact:

- Immediately remove material using cotton soaked with alcohol and wash thoroughly with soap and water.

Eye contact:

- Rinse immediately with copious amounts of clean water and seek medical attention.

Ingestion:

- Seek immediate medical assistance.

If skin rash, irritation, sensitization, or other allergic reactions occur, discontinue use and seek medical advice.

PRECAUTIONS

- Product intended for professional dental use only
- Wear appropriate personal protective equipment including gloves, mask, and protective eyewear during handling
- Use rubber dam isolation during clinical procedures whenever possible
- Avoid direct contact with skin, eyes, and oral soft tissues
- Use minimal quantity necessary to achieve the intended clinical effect
- Avoid excessive placement of material into surrounding tissues

ADVERSE REACTIONS

- Possible adverse reactions may include:
- Local tissue irritation
- Hypersensitivity reactions
- Burning sensation
- Allergic response in sensitive individuals

DIRECTIONS FOR USE

- Isolate the treatment area appropriately.
- Prepare the cavity or root canal according to standard clinical procedures.
- Soak a sterile cotton pellet with a small quantity of Prevalent Formacresol.
- Place the soaked cotton pellet into the pulp chamber or canal area as indicated.
- Seal the cavity temporarily.
- Remove the dressing after approximately one week or according to clinical judgment.
- Complete the required treatment and restore the tooth appropriately.

Important:

- Use only a minimal amount of medicament.
- Avoid direct contact with periapical tissues.
- Do not allow excess solution to overflow.

PRESENTATION

Ref. No.: PL-0107- Formacresol- 1 x 15 ml Bottle

STORAGE CONDITIONS

- Store at temperatures between 10°C and 24°C
- Keep container tightly closed after use
- Store in a cool, dry place away from direct sunlight and moisture
- Do not use after the expiry date
- Shelf life remains unaffected after opening if stored according to instructions

SHELF LIFE

3 years from date of manufacturing

DISPOSAL

Dispose of product, packaging, and contaminated materials according to local regulations and applicable biomedical waste disposal requirements.


CONTACT


 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com



Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer. This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel


 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP DRY"



 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"


 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"


 Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "CAUTION"


 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "CONSULT INSTRUCTION FOR USE"

 Symbol for "CATALOGUE NUMBER"  Symbol for "USE BY"

 Symbol for "MEDICAL DEVICE"

 Symbol for "NON STERILE"

 Symbol for "BATCH CODE"

Document No	PL-IFU-065
Revision	02
Date	10-05-2025