

INSTRUCTION FOR USE

GP Solvent

(Gutta Percha Solvent)

Prevalent GP Solvent is a D-Limonene-based solvent used during endodontic retreatment procedures for softening and removing gutta-percha and Zinc Oxide–Eugenol based root canal filling materials. It can also be used for cleaning endodontic instruments contaminated with root canal filling materials. The formulation is based on naturally derived components and is designed for clinical use under oral conditions.

COMPOSITION

D-Limonene

PROPERTIES

- Effective softening and removal of gutta-percha during retreatment procedures
- Assists in dissolving Zinc Oxide–Eugenol based root canal filling materials
- Can be used as a cleaning agent for endodontic instruments
- Naturally derived formulation
- Suitable for use in the oral environment

INDICATIONS

- Removal of gutta-percha during root canal retreatment
- Removal of Zinc Oxide–Eugenol based root canal filling materials
- Cleaning of endodontic instruments contaminated with root canal filling materials

CONTRAINDICATIONS

- Do not use in patients with known hypersensitivity or allergy to D-Limonene or any component of the product
- Do not use in patients with a history of severe allergic reactions to citrus-derived compounds
- Do not use for applications other than intended use

WARNINGS

- Product intended for professional dental use only
- Prevalent GP Solvent is volatile and flammable
- Keep away from heat, sparks, open flames, and ignition sources
- Avoid inhalation of vapors
- Avoid prolonged or repeated contact with skin and eyes
- Do not ingest the product
- Keep out of reach of children

In case of accidental exposure:

Skin contact:

- Wash thoroughly with soap and water.

Eye contact:

- Flush immediately with copious amounts of clean water and seek medical attention.

Inhalation:

- Move the individual to fresh air if irritation occurs.

Ingestion:

- Seek immediate medical assistance.

PRECAUTIONS

- Wear appropriate personal protective equipment including gloves, masks, and protective eyewear during use
- Use rubber dam isolation whenever possible during clinical procedures
- Use only the minimum amount required for the procedure
- Keep bottle tightly closed immediately after use to minimize evaporation
- Avoid contamination of the product during dispensing

ADVERSE REACTIONS

- No known adverse effects have been reported when the product is used according to instructions and recommended storage conditions.
- Possible reactions may include:
 - Local tissue irritation
 - Allergic reactions in sensitive individuals
 - Skin sensitization following repeated exposure

DIRECTIONS FOR USE

- Dispense the required amount of Prevalent GP Solvent into a clean dappen dish or onto a sterile cotton pellet.
- Apply the solvent directly onto the root canal filling material to be removed.
- Allow sufficient contact time for softening of the material.
- Remove the softened filling material using suitable endodontic instruments.
- Repeat the procedure if required until complete removal is achieved.
- For instrument cleaning:
 - Apply the solvent onto contaminated endodontic instruments.
 - Remove softened material and clean instruments according to standard sterilization procedures.
- Important:
 - Use only the minimum quantity necessary.
 - Avoid excessive application beyond the intended treatment area.

PRESENTATION

Ref. No.: PL-0112- GP Solvent- 1 x 15 ml Bottle

STORAGE CONDITIONS

- Store at temperatures between 5°C and 25°C
- Keep container tightly closed immediately after use
- Store in a cool, dry, and well-ventilated place
- Protect from heat, direct sunlight, and ignition sources
- Do not use after the expiry date


SHELF LIFE

3 years from the date of manufacture

DISPOSAL














Dispose of unused product, packaging materials, and contaminated waste according to local regulations and environmental guidelines applicable to chemical and medical waste.

CONTACT

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.
Ph: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer. This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "TEMPERATURE LIMITATION"
-  Symbol for "KEEP DRY"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"
-  Symbol for "CAUTION"
-  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "CATALOGUE NUMBER"
-  Symbol for "USE BY"
-  Symbol for "MEDICAL DEVICE"
-  Symbol for "NON STERILE"
-  Symbol for "BATCH CODE"

Document No	PL-IFU-015
Revision	02
Date	10-08-2025