

## INSTRUCTION FOR USE

### Preva Cure

#### (Dry Socket Dressing Paste)

Preva Cure is an antiseptic and analgesic alveolar dressing paste intended for the management of alveolar osteitis following tooth extraction. The formulation combines a fibrous carrier, antiseptic ingredients, and soothing agents to provide temporary pain relief, protect the extraction socket, and support the natural healing process.

#### COMPOSITION

Penghawar Djambi, Olive Oil, Calcium Carbonate, Iodoform, Eugenol, Spearmint Oil, Pharmaceutical-grade Excipients

#### INDICATIONS

- Management of dry socket (alveolar osteitis)
- Temporary dressing of painful extraction sockets
- Relief of discomfort associated with exposed alveolar bone following tooth extraction
- Protection of extraction sites from food debris and external irritants
- Temporary palliative treatment following dental extraction as determined by a dental professional

#### CLINICAL BENEFITS

- Provides soothing relief from pain and discomfort associated with dry socket.
- Delivers antiseptic action within the extraction socket.
- Forms a protective dressing over the affected area.
- Helps reduce irritation caused by food particles and external contaminants.
- Supports patient comfort during the healing process.

#### CONTRAINDICATIONS

Do not use:

- In patients with known hypersensitivity to eugenol, clove oil, iodine-containing compounds, iodoform, or any component of the product.
- In patients with a history of allergic reactions to essential oils.
- In cases requiring immediate surgical management.
- In extraction sites exhibiting severe infection requiring systemic treatment, unless directed by a dental professional.

#### WARNINGS

- For professional dental use only.
- Not intended for permanent implantation.
- Do not overfill or tightly pack the extraction socket.
- Avoid ingestion of the product.
- Keep out of reach of children.
- Avoid contact with eyes and non-target tissues.

#### PRECAUTIONS

- Use only under the supervision of a qualified dental professional.
- Employ aseptic techniques during application.
- Reassess the patient periodically and replace or remove the dressing as clinically indicated.
- Use with caution in patients with known iodine sensitivity.
- Discontinue use if signs of hypersensitivity or irritation occur.

#### DIRECTIONS FOR USE

1. Examine the extraction socket and confirm the diagnosis of dry socket (alveolar osteitis).
2. Gently irrigate the socket with sterile saline or according to standard clinical protocol.
3. Remove loose debris without disturbing healthy granulation tissue.
4. Isolate and gently dry the treatment area.
5. Using a sterile instrument, place a small amount of Preva Cure into the socket.
6. Adapt the dressing lightly to the contours of the socket without exerting pressure.
7. Ensure patient comfort and stability of the dressing.
8. Review the patient clinically and replace or remove the dressing as necessary.

#### ADVERSE REACTIONS

Possible adverse effects may include:

- Localized irritation
- Transient burning sensation
- Allergic or hypersensitivity reactions
- Temporary taste alteration
- Delayed healing associated with prolonged retention or excessive packing

If adverse reactions occur, discontinue use and manage appropriately.

#### PRESENTATION

Ref. No: PL-0269- Preva Cure- 15g Jar

#### STORAGE

- Store between 10°C and 30°C.
- Keep the container tightly closed after use.
- Protect from direct sunlight, heat, and moisture.
- Do not freeze.

#### SHELF LIFE

3 years from date of manufacturing.

#### DISPOSAL

Dispose of unused product and contaminated materials in accordance with local regulations for healthcare and clinical waste.



#### CONTACT

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala - 679339, India.  
Ph: +91 4931257954 Email: info@prevalentonline.com  
www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.



This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.


**This product is specially formulated for use in dentistry and to be used by qualified dental personnel**


 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP DRY"



 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"


 Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"


 Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "CAUTION"


 Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "CONSULT INSTRUCTION FOR USE"

 Symbol for "CATALOGUE NUMBER"  Symbol for "USE BY"

 Symbol for "MEDICAL DEVICE"

 Symbol for "NON STERILE"

 Symbol for "BATCH CODE"

Document No	PL-IFU-044
Revision	02
Date	10-05-2025