

INSTRUCTION FOR USE

PrevaNecro

(Non Arsenic Pulp Devitalizer Paste)

Prevalent's PrevaNecro Pulp Devitalizer Paste is a non-arsenic devitalizing material intended for devitalization of dental pulp prior to endodontic treatment procedures. The formulation is designed to assist pulp devitalization while providing local anesthetic support and antibacterial action.

COMPOSITION

Racemic Camphor, Polyoxymethylene, Para-Chlorophenol, Lidocaine Hydrochloride, Paste-forming component, Colorant, Fiber filler

INDICATIONS FOR USE

- Devitalization of vital dental pulp before root canal treatment
- Cases where conventional anesthesia is difficult or insufficient
- Temporary pulp devitalization prior to pulp extirpation
- Management of pulp tissue before endodontic procedures

CONTRAINDICATIONS

Do not use in:

- Patients with known hypersensitivity or allergy to any component of the material
- Teeth with necrotic pulp
- Primary teeth unless specifically recommended by the clinician
- Pregnant or breastfeeding patients unless the benefits outweigh potential risks
- Patients with known hypersensitivity to lidocaine or related anesthetics
- Cases with severe periapical lesions or extensive periodontal destruction

PROPERTIES

- Arsenic-free formulation
- Assists pulp devitalization before endodontic procedures
- Contains local anesthetic component for patient comfort
- Contains antimicrobial ingredients
- Easy handling and placement
- Distinct coloration for visibility during application

DIRECTIONS FOR USE

- Isolate the tooth using an appropriate isolation method, preferably a rubber dam.
- Prepare the cavity and expose the pulp chamber according to standard clinical procedures.
- Remove debris and gently dry the area without excessive desiccation.
- Apply a small amount of PrevaNecro paste directly over the exposed pulp tissue.
- Avoid contact with surrounding soft tissues.
- Cover the paste with a sterile cotton pellet if required.
- Seal the cavity temporarily using an appropriate temporary restorative material.
- Recall the patient after approximately 24–48 hours, or according to clinical judgment.

- Remove the temporary restoration and paste completely before proceeding with pulp extirpation and root canal treatment.

WARNINGS

- For professional dental use only.
- Do not allow the material to contact oral soft tissues, gingiva, or mucosa.
- Avoid excessive placement of material.
- Ensure proper temporary sealing to avoid leakage.
- Prolonged placement beyond the recommended period may increase the risk of irritation.
- Do not swallow the material.
- Keep away from children.

PRECAUTIONS

- Use only the minimum amount necessary.
- Avoid extrusion beyond the pulp chamber.
- Use with caution in patients with a history of allergic reactions.
- Use personal protective equipment during handling.
- Follow standard infection-control procedures.

ADVERSE REACTIONS

- Temporary local irritation
- Sensitivity reactions
- Allergic response to ingredients
- Soft tissue irritation if accidental contact occurs
- Postoperative discomfort
- Discontinue use and seek appropriate medical attention if severe reactions occur.

PRESENTATION

Ref.No: PL-0159- PrevaNecro- 1 x 3g Syringe

STORAGE CONDITIONS

- Store at temperatures between 5°C–30°C
- Keep tightly closed after use
- Protect from direct sunlight and excessive heat
- Do not use after expiration date


SHELF LIFE

3 years from date of manufacture

DISPOSAL

Dispose of unused material and packaging according to local regulations for medical and dental waste.












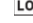

CONTACT

 **Prevalent Healthcare LLP**, Poongode Road, Karutheni, Malappuram, Kerala – 679339, India.
Ph.: +91 4931257954 Email: info@prevalentonline.com
www.prevalentonline.com

Failure to comply with the specified storage conditions may alter the working characteristics of the material and reduce its shelf life. The manufacturer is not responsible for any loss of quality resulting from non-compliance with the specified conditions for transportation, storage, and use of this product. The user assumes full responsibility for using the material for any purposes other than those specified by the manufacturer.

This material is intended solely for professional dental use. Application must be carried out strictly according to the instructions for use. The manufacturer accepts no liability for damages resulting from failure to follow the provided instructions or for use beyond the stipulated applications. The user is responsible for assessing the material's suitability for any purpose not explicitly stated in this instruction sheet. The descriptions and data provided do not constitute a warranty of attributes and are not legally binding.

This product is specially formulated for use in dentistry and to be used by qualified dental personnel

-  Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP DRY"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
-  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "CAUTION"
-  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "CONSULT INSTRUCTION FOR USE"
-  Symbol for "CATALOGUE NUMBER"  Symbol for "USE BY"
-  Symbol for "MEDICAL DEVICE"
-  Symbol for "NON STERILE"
-  Symbol for "BATCH CODE"

Document No	PL-IFU-025
Revision	02
Date	10-08-2025